

Single Antigen Naming Convention when Combining Groups 1, 2 and 3 for Analysis

When importing a .csv file where multiple groups of the LABScreen Single Antigen have been combined for sample processing (LS1A01 + LS1A02, LS1A01 + LS1A03, LS1A02 + LS1A03, or LS1A01 + LS1A02 + LS1A03), the user is required to designate a composite catalog ID. Because the groups being combined could potentially have different lot numbers, the following naming convention has been developed to provide the user with all possible lot # combinations. If the combination needed is not indicated in the drop-down menu upon import, this means that the lot #'s in question cannot be combined.

The catalog ID's and Lot number combinations of two groups may include one of four alphanumeric characters. The absence of an alphanumeric character indicates that the lot number is the same for both groups.

A = The lot as written for group one and the lot number as written plus one for group 2
B = The lot as written for group one and the lot number as written plus two for group 2
Z = The lot as written for group one and the lot number as written minus one for group 2
Y = The lot as written for group one and the lot number as written minus two for group 2

Please refer to the following examples:

LS1A1&2 002 - designates LS1A01 Lot #002 + LS1A02 Lot #002
LS1A1&2 02A - designates LS1A01 Lot #002 + LS1A02 Lot #003
LS1A1&2 02Z - designates LS1A01 Lot #002 + LS1A02 Lot #001
LS1A1&2 06Z - designates LS1A01 Lot #006 + LS1A02 Lot #005
LS1A2&3 05A - designates LS1A02 Lot #005 + LS1A03 Lot #006

The catalog ID's and Lot number combinations of three groups consist of the character "C" plus three numbers which designate the lot numbers for each group.

LS1A123 C556 - designates LS1A01 Lot #005 + LS1A02 Lot #005 + LS1A03 Lot #006
LS1A123 C656 - designates LS1A01 Lot #006 + LS1A02 Lot #005 + LS1A03 Lot #006
LS1A123 C655 - designates LS1A01 Lot #006 + LS1A02 Lot #005 + LS1A03 Lot #005